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
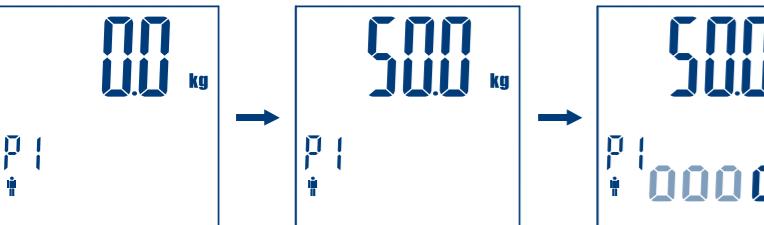
Daily Measurement
The Pic BodyStation scale will switch on as you step on the platform barefooted. Stand still and keep full contact with the electrodes until the display stops displaying moving "0".

Warning! Check every time the user is correctly identified by the scale.

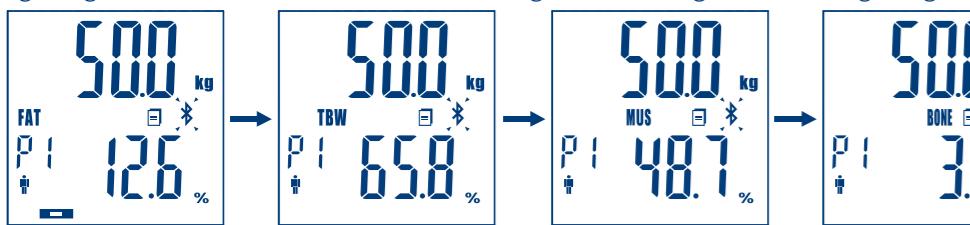According to the analysis results, the system will automatically identify the possible User ID with most similar history records. Then the measuring results will be displayed sequentially two times. And meanwhile the indicator light lights with different color. Compared with the last record of the same user ID, when the current weight is increased over 0.5kg, the red light lights; when it is reduced over 0.5kg, the green light lights; when it is the same (in the range of +/-0.5kg), the blue light lights.

When the system finds out two or more users with similar history records, it will notify you to choose between, for example, P1 and P2. You may press **◀** key for P1 and/or **▶** key for P2. The measuring results will then be sorted into the User ID you selected and displayed sequentially two times.

P1P2

If it fails to identify the possible User ID, the display will only display the weight data. (To find out the solutions, please refer to Troubleshooting for more details.)

When your scale is successfully paired with your smartphone and the Bluetooth® connection is ON, Pic BodyStation will process data transmission instead. (Please refer to Data Transmission for more details.)

Data Transmission

Once the Pic BodyStation scale is successfully paired-up with your smartphone, please open the Pic Health Station App and turn on your device Bluetooth® connection. Keep smartphone and scale close. Step on the scale and take a measurement. Once the measurement is finished the data is automatically transferred to the app.

ONLY when the measuring results are attached to a specific User ID will it be transmitted to your mobile after measurement.

The symbol will disappear after successful data transmission, and you may check your personal health data stored in your smartphone.

If the data transmission fails, the symbol will remain. The pending measurement data will be temporarily kept in the scale and transmitted to your smartphone when next measurement is complete.

- Data transmitting
- Successful Bluetooth® connection
- Data transmitting to the APP

- If SUCCEED, the symbol disappears;
- If FAIL, the symbol remains.

How to mitigate possible interference?

- The range between the scale and the smartphone should be reasonably close, from 1 meter to 10 meters. Please ensure no obstacles between the scale and the smartphone so as to obtain quality connection.
- To avoid interference, other electronic devices (particularly those with Bluetooth® transmission / Transmitter) should be kept at least 1 meter away from the scale.

Error Prompt

Error	Description	Solution
	Overload. The device will power off.	Stop using this scale for measurement.
	Low Battery. The device will power off after about 4s.	Replace all four batteries in the same time. Please purchase the authorized batteries for replacement.
	Failure of pairing up your scale with your smartphone.	Please check the below items: - Bluetooth® connection is ON - App is ON - Both devices are within the transmission distance

When Measuring:

Problem	Root Cause	Solution
Abnormal measuring results: - Too high; or - Too low; or - Huge difference between two recent measurement.	Incorrect posture The device is located on the soft ground such as a carpet or on a rugged surface.	Please step on the platform barefooted and stand still. Place the scale on a hard, flat and stable surface.
Abnormal measuring results: - Too high; or - Too low; or - Huge difference between two recent measurement	Your feet are too dry.	Wipe your feet with a damp cloth, keeping them slightly damp when starting measurement.
The display doesn't turn ON	Batteries not yet installed.	Install the batteries. (Please refer to Insert the Batteries)
	Worn batteries.	Replace all four batteries at the same time. Please purchase the authorized batteries for replacement.
CANNOT proceed to analyze body fat, total body water, muscle mass and bone mass.	Step onto the platform wearing socks or shoes.	Please keep barefooted during the measurement, and keep full contact with the platform as well.
	The system cannot identify the possible User ID with most similar data.	Please assign a User ID following the instruction in Set Up Your Profile.
	The user fails to select the User ID from what the system found.	Please assign a User ID following the instruction in Set Up Your Profile.
The device powers off.	Low battery.	Replace all four batteries at the same time. Please purchase the authorized batteries for replacement.

When Data Transmitting:

Problem	Root Cause	Solution
Data transmission failed.	Smartphone Bluetooth® connection is OFF. The APP is OFF. Out of range of Bluetooth® transmission. None of the user ID is assigned. Smartphone is in standby/power saving mode. Activate the smartphone or exit the power saving mode.	Turn ON the Bluetooth® connection via Settings. Press the icon to run your APP. Place your smartphone closer to the scale. Please assign a User ID following the instruction in Set Up Your Profile.

SPECIFICATIONS

Product Name:

Pic BodyStation

Model: 09194

REF: 02009194000000

Dimension Scale: 307x307x24mm (Approximately)

Net Weight: Approximately 1.75kg (Excluding the dry cells)

Display: Digital Screen

Measurement Unit: Kilogram/Stone/Pound

Measurement Range: 5kg to 180kg

Division: 0.1kg / 0.2lb / st.0.2lb

Accuracy: 50kg: ±0.3kg;

100kg: ±0.4kg;

150kg: ±0.5kg;

180kg: ±0.7kg

Working Environment:

Temperature: 5°C to 40°C
Relative Humidity: ≤90% RH
Atmospheric pressure: 86kPa to 106kPa
Temperature: -20°C to 60°C
Atmospheric pressure: 50kPa to 106kPa
Relative Humidity: ≤90% RH

Storage Environment:

Power Source: 6V (4 X 1.5V LR03 AAA-size batteries)

Turn on Method: Auto switch-on

Auto-OFF: About 10s after the screen displays "0.0"

Accessories: 1) 4 X 1.5V LR03 (AAA-size) batteries

2) User Manual

Mode of Operation: Continuous Operation

Frequency Range: 2.402 - 2.480 GHz

Output Power Range: ≤ 4.74 dBm

Declaration of EU Compliance:

Hereby, PIKDARE S.p.A. declares that the radio equipment type Mod.09194, is in compliance with Directive 2014/53/EU and directive 2011/65/EU (RoHS). The full text of the EU declaration of conformity is available at the following internet address: <http://web.picsolution.com/pichealthstation/en/pic-body-station/it.html>

About the Accuracy of This Product

This product passes strict inspection before delivery and therefore its accuracy is guaranteed by the manufacturer.

Please refer to the above table for the descriptions on accuracy.

This product is specially designed for body fat analysis as well as weight measurement. It should NOT be used by anyone during the process of transaction for verification of goods' weight.

Maintenance

When carrying out usual maintenance, please ensure practice of the following Do's

and Don'ts:

- DO use a dry soft cloth to wipe the dust.
- DO use a wet soft cloth, dipped into water and wrung out, to wipe the dirt. Then use a dry soft cloth to dry up the device.
- DON'T wash the device with water or immerse it in water.
- DON'T use propellant, abrasive or other chemicals to wipe the dirt in avoidance of discolor or malfunction.

Table of Body Fat Level (Unit: %)

a) The body fat percentage (%): 5%-60%/0.1%

Standard for Men

Rating	Age				
	20-29	30-39	40-49	50-59	60+
Low	<13	<14	<16	<17	<18
Normal	13-20	14-21	16-23	17-24	18-25
Moderately High	20.1-23	21.1-24	23.1-26	24.1-27	25.1-28
High	>23	>24	>26	>27	>28

Standard for Women

Rating	Age				
	20-29	30-39	40-49	50-59	60+
Low	<19	<20	<21	<22	<23
Normal	19-28	20-29	21-30	22-31	23-32
Moderately High	28.1-31	29.1-32	30.1-33	31.1-34	32.1-35
High	>31	>32	>33	>34	>35

Source: University of Illinois Department of Food Science and Human Nutrition. Body Fat Percentage Calculator.

Table of Body Water Level (Unit: %)

b) The body water percentage (%): 43%-73%/0.1%

Source: Derived from Wang & Deurenberg: "Hydration of fat-free body mass". American Journal Clin Nutr 1999;69:833-841.

	BF % RANGE	OPTIMAL TBW % RANGE
Men	4 to 14%	70 to 63%
	15 to 21%	63 to 57%
	22 to 24%	57 to 55%
	25 and over	55 to 37%
Women	4 to 20%	70 to 58%
	21 to 29%	58 to 52%
	30 to 32%	52 to 49%
	33 and over	49 to 37%

Muscle Mass Percentage

(Source: International Commission on Radiological Protection, 1975)

Men: Approximately 40% of total body weight

Women: Approximately 30% of total body weight

Bone Mass Percentage

(Source: Rico et al.1993)

The average bone mass percentage for both men and women is between 4 to 5%.

Health Tips - About Body Fat

Fat is essential for human body. It can not only store energy and protect viscera, but also regulate body temperature and maintain normal physiological function of human body.

However, too much body fat is harmful to human body. It is always accompanied by Fatty Liver, diabetes, coronary heart disease, etc.

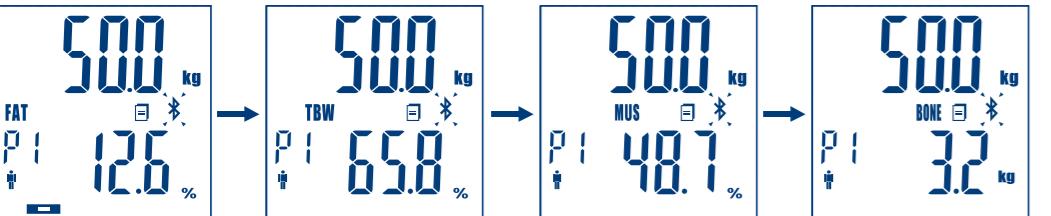
Therefore self-measuring and self-monitoring body fat level are beneficial to your health.

Since we can't judge body fat level simply by our weight, this Pic BodyStation scale, with BIA (Bio-impedance Analysis) technology applied, is an accurate device that offers a quick and comfortable way to obtain your body fat level.

Guidance and manufacturer's declaration – electromagnetic emissions

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.

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